DISPOSABLE CYTOLOGY BRUSH
BROSSES CYTOLOGIQUES JETABLES
CEPILLOS DE CITOLOGIA
DESECHABLES

EN - INSTRUCTION FOR USE	2
FR - MODE D'EMPLOI	4
ES - INSTRUCCIONES DE USO	6

G-Flex Europe S.P.R.L. - 20, Rue de l'industrie 1400
Nivelles - Belgium | **Phone:** +32 67.88.36.65
Email: info@g-flex.com | **Website:** www.g-flex.com
US Commercial Office Only:
G-Flex USA, Inc., 74 Southwood Pkwy Suite 600
Atlanta, GA 30354 – USA

 **0120**

EN - INSTRUCTION FOR USE

Intended Use

Disposable Cytology Brushes are used for taking specimens from the human body during endoscopic examinations.

Product Description

The device is composed of a brush head, stainless steel wire, outer plastic tube, slider and operating handle. It is available in different lengths.

Advices before first application

Please read carefully and follow all safety operating instructions and warnings before first application of the device.

A previous knowledge regarding handling and operation is required and essential.

Unpack the instrument carefully and examine the devices for any possibility of damage. In case of any damage or missing items contact your distributor immediately.

If the package present any damaged, the sterility of the device is not guarantee.

Handling and Operation

The Cytology Brush has to be retracted completely into the teflon tube to be introduced into the operating channel. Please observe the compatibility between the Cytology Brush diameter and the endoscopes operating channel diameter.

Procedure

- Take the brush completely out of the tube by pushing the handle.
- Place the brush in front of the tissue area to be analyzed and swab it.
- Slide the brush completely back into the tube by pulling the handle.
- Remove the Cytology Brush carefully out of the endoscope.

Storage

This device is delivered sterilized and in order to maintain this state until its first use it must be kept in its original sterilized package.

Do not put any objects on the instrument or its package!
Do not store the instruments near aggressive chemical products!

Do not expose the instruments to direct or indirect sunlight or other ultra-violet rays!

Keep in dry area

Complaints will not be taken under consideration if the instruments have been stored improperly.

Cleaning and Sterilization

This device is designed and warranted for single use!

Risk in case of reuse

This device is for single use only. G-Flex denied all responsibility in case of reuse. The reuse of single use device might present risk to the safety of the user or patient, due to possible uncontrolled contamination and / or unreliability of the device performance.

User

The users of G-FLEX instruments must be specialists in their fields. An appropriate and specific training for preparation, care and maintenance of the flexible instruments is required.

Repairing and complains

In health protection of our employees only disinfected or sterilized instruments will be accepted for analysis or repairing. In any case disinfection or sterilization dates and validity must be labelled outside the package. If this requirement has not been fulfilled the instruments will be returned to the sender without analysis or repairing.

Legal foundation

The Law of the European Union applied.

Product support

In case of questions or difficulties concerning our instruments please contact your local distributor or G-FLEX directly during regular working hours.

Times available: Monday to Friday; 9am to 5pm (MET)

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