**DISPOSABLE EXTRACTION BASKETS  
PANIER D'EXTRACTION JETABLE  
WEGWERP-EXTRACTIEMANDJES  
CESTA DE EXTRACCIÓN  
DESECHABLE**

EN - INSTRUCTION FOR USE .....	2
FR - MODE D'EMPLOI .....	4
NL - GEBRUIKSAANWIJZING .....	6
ES - INSTRUCCIONES DE USO .....	8

**G-Flex Europe S.P.R.L.** - 20, Rue de l'industrie 1400  
Nivelles - Belgium | **Phone:** +32 67.88.36.65  
**Email:** [info@g-flex.com](mailto:info@g-flex.com) | **Website:** [www.g-flex.com](http://www.g-flex.com)  
**US Commercial Office Only:**  
G-Flex USA, Inc., 74 Southwood Pkwy Suite 600  
Atlanta, GA 30354 – USA

# EN - INSTRUCTION FOR USE

## Intended Use

Disposable Extraction Baskets are intended to catch and remove stones from the biliary duct of the human body. Contrast injection is possible through the lateral Luer-Lock connector.

## Product Description

The device is made of a basket at the distal end (multi-braided or nitinol) and a handle at the proximal end. The handle can be removable depending on the model.

## Advices before first application

Please read carefully and follow all safety operating instructions and warnings before first application of the device.

A previous knowledge regarding handling and operation is required and essential. Unpack the instrument carefully and examine the devices for any possibility of damage. In case of any damage or missing items contact your distributor immediately.

If the package present any damaged, the sterility of the device is not guarantee.

## Handling and Operation

The device must be passed through the endoscope channel with its basket withdrawn into the tube. Please observe compatibility between the device and the endoscope operating channel diameters.

## Procedure

### Extraction of Stone

- Once the stone is located by usual means do as following:
- Place the Extraction Basket's distal tip approx. 20 mm in front of the stone to be removed.
- Push the basket completely out of the tube and bring the stone in the basket centre.
- Pull the handle back until the stone is fixed in the basket and pull the device toward of the endoscope until the stone is extracted.

Excessive traction should be avoided to prevent damage the endoscope.

### Emergency Lithotripsy

- If the stone (because of its dimension) cannot be removed and it cannot be loosened out of the basket, then the stone need to be crushed with the use of an emergency lithotripter (see lithotripter instruction for use for more detail on this procedure).

-  A lithotripsy has no guarantee of total success! If the stone to be crushed is very hard the basket might be torn because of the force applied on it. In this case the stone along with the torn basket should be surgically removed.
-  The lithotripsy should be interrupted immediately if the stone crushing could not be achieved.

## Storage

This device is delivered sterilized and in order to maintain this state until its first use it must be kept in its original sterilized package.

-  Do not put any objects on the instrument or its package!  
Do not store the instruments near aggressive chemical products!

-  Do not expose the instruments to direct or indirect sunlight or other ultra-violet rays!

-  Keep in dry area

Complaints will not be taken under consideration if the instruments have been stored improperly.

## Cleaning and Sterilization

-  This device is designed and warranted for single use!

## Risk in case of reuse

This device is for single use only. G-Flex denied all responsibility in case of reuse. The reuse of single use device might present risk to the safety of the user or patient, due to possible uncontrolled contamination and / or unreliability of the device performance.

## User

The users of G-FLEX instruments must be specialists in their fields. An appropriate and specific training for preparation, care and maintenance of the flexible instruments is required.

## Repairing and complains

In health protection of our employees only disinfected or sterilized instruments will be accepted for analysis or repairing. In any case disinfection or sterilization dates and validity must be labelled outside the package. If this requirement has not been fulfilled the instruments will be returned to the sender without analysis or repairing.

## Legal foundation

The Law of the European Union applied.

## Product support

In case of questions or difficulties concerning our instruments please contact your local distributor or G-FLEX directly during regular working hours.

**Times available:** Monday to Friday; 9am to 5pm (MET)

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