**DISPOSABLE HOT BIOPSY FORCEPS**  
**FORCEPS DE BIOPSIE THERMIQUE JETABLE**  
**FORCEPS DE BIOPSIA CALIENTE DESECHABLE**

EN - INSTRUCTION FOR USE .....	2
FR - MODE D'EMPLOI .....	5
ES - INSTRUCCIONES DE USO .....	8

**G-Flex Europe S.P.R.L.** - 20, Rue de l'industrie 1400  
Nivelles - Belgium | **Phone:** +32 67.88.36.65  
**Email:** [info@g-flex.com](mailto:info@g-flex.com) | **Website:** [www.g-flex.com](http://www.g-flex.com)  
**US Commercial Office Only:**  
G-Flex USA, Inc., 74 Southwood Pkwy Suite 600  
Atlanta, GA 30354 – USA

 **0120**

# EN - INSTRUCTION FOR USE

## Intended Use

Disposable Hot Biopsy Forceps are used for taking out biopsy specimens (tissues samples) from the human body, through endoscopes.

## Product Description

The device is composed by a jaw, a spiral spring hose and an operating handle with HF connector.

## Advices before first application

Please read carefully and follow all safety operating instructions and warnings before first application of the device.

A previous knowledge regarding handling and operation is required and essential.

Unpack the instrument carefully and examine the devices for any possibility of damage. In case of any damage or missing items contact your distributor immediately.

If the package present any damaged, the sterility of the device is not guarantee.

## Handling and Operation

The device can only be used with an endoscope, a high frequency (HF) generator and an active cord.

## Insertion into the Endoscope

The device must be passed through the endoscope operating channel with its snare withdrawn into the tube. Please observe compatibility between the device and the operating channel diameters.

## HF active cord Connection

Connect the active cord to electricity connector at the handle and to the HF-generator.

## HF Generators

The device can be connected to all licensed HF generators by using the active cord. Always check compatibility between HF generator and active cord. In doubt contact the manufacturer of generator or your local distributor.

## Procedure

- Place the Hot Biopsy Forceps near to the tissue to be removed out of the patient.
- Open the cups by pushing the plastic spool at the handle.
- Place the Hot Biopsy Forceps into the tissue and close the cups at the forceps by pulling back the spool.

- Apply the electric current on the hot biopsy forceps (Please observe the HF source manufacturer's instructions).
- Gently take out the tissue sample.
- Stop HF electric current and keep the forceps cups closed.
- Take the Hot Biopsy Forceps completely out of the endoscope with its cups closed.
- This tissue specimen taken will be ready for further histological examination.

Do not touch another tissue areas when using HF electricity with the forceps cups (high risk of hurt for the patient). Do not use the rinse adapter during the HF working, but only for cleaning.

## Storage

This device is delivered sterilized and in order to maintain this state until its first use it must be kept in its original sterilized package.

Do not put any objects on the instrument or its package!  
Do not store the instruments near aggressive chemical products!

Do not expose the instruments to direct or indirect sunlight or other ultra-violet rays!

Keep in dry area

Complaints will not be taken under consideration if the instruments have been stored improperly.

## Cleaning and Sterilization

This device is designed and warranted for single use!

## Risk in case of reuse

This devices is for single use only. G-Flex denied all responsibility in case of reuse. The reuse of single use device might present risk to the safety of the user or patient, due to possible uncontrolled contamination and / or unreliability of the device performance.

## User

The users of G-FLEX instruments must be specialists in their fields. An appropriate and specific training for preparation, care and maintenance of the flexible instruments is required.

## Repairing and complains

In health protection of our employees only disinfected or sterilized instruments will be accepted for analysis or repairing. In any case disinfection or sterilization dates

and validity must be labelled outside the package. If this requirement has not been fulfilled the instruments will be returned to the sender without analysis or repairing.

## **Legal foundation**

The Law of the European Union applied.

## **Product support**

In case of questions or difficulties concerning our instruments please contact your local distributor or G-FLEX directly during regular working hours.

**Times available:** Monday to Friday; 9am to 5pm (MET)

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