**DISPOSABLE FOREIGN BODY
RETRIEVERS
L'EXTRACTEUR DE CORPS
ÉTRANGERS JETABLE
WEGWERPINSTRUMENTEN OM
CORPORA ALIENA TE VERWIJDEREN
FÓRCEPS PARA CUERPO EXTRAÑO
DESECHABLES**

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EN - INSTRUCTION FOR USE

Intended Use

Disposable Foreign Body Retrievers are intended for taking out foreign bodies from the human body through endoscopes.

Product Description

The Foreign Body Retrievers are made of a flexible tube with a grasping distal end controlled by a handle.

Advices before first application

Please read carefully and follow all safety operating instructions and warnings before first application of the device.

A previous knowledge regarding handling and operation is required and essential.

Unpack the instrument carefully and examine the devices for any possibility of damage. In case of any damage or missing items contact your distributor immediately.

If the package present any damaged, the sterility of the device is not guarantee.

Handling and Operation

Disposable Foreign Body Retrievers must be inserted through the endoscope operating channel with its graspers withdrawn into the tube. Please observe compatibility between Foreign Body Retrievers and the endoscope operating channel diameter.

Foreign Body Retrievers are indicated for suspected gastritis, diagnosis (color patches, Mucosa defect, tissue bulges Bever) and Chronic diarrhea

Procedure

Once the foreign body is located in the patient's body the following procedure should be done:

- Insert the device into the endoscope.
- Place the distal end of tube in front of the object to be removed.
- Push the handle spool in order to take the grasping end out of the tube and then bring the object to be captured in the center of the graspers.
- Close the prongs carefully as much as the object allows by pulling back the handle spool.
- Remove the object and the grasper gently withdrawing the endoscope. Always keep the grasper prongs tightly closed.

Contraindication

- Bleeding disorders
- Tissue samples from stumps in vascular ulcers
- Inadequate preparation of the patient (intestinal)

- General. OGD, colonoscopy, proctoscopy, sigmoidoscopy

Possible complication

- Perforation, hemorrhage, infection, sepsis, allergic reaction to medication, hypotension, respiratory depression or arrest, cardiac arrhythmia or arrest.

Storage

This device is delivered sterilized and in order to maintain this state until its first use it must be kept in its original sterilized package.

Do not put any objects on the instrument or its package!
Do not store the instruments near aggressive chemical products!

Do not expose the instruments to direct or indirect sunlight or other ultra-violet rays!

Keep in dry area

Complaints will not be taken under consideration if the instruments have been stored improperly.

Cleaning and Sterilization

This device is designed and warranted for single use!

Risk in case of reuse

This device is for single use only. G-Flex denied all responsibility in case of reuse. The reuse of single use device might present risk to the safety of the user or patient, due to possible uncontrolled contamination and / or unreliability of the device performance.

User

The users of G-FLEX instruments must be specialists in their fields. An appropriate and specific training for preparation, care and maintenance of the flexible instruments is required.

Repairing and complains

In health protection of our employees only disinfected or sterilized instruments will be accepted for analysis or repairing. In any case disinfection or sterilization dates and validity must be labelled outside the package. If this requirement has not been fulfilled the instruments will be returned to the sender without analysis or repairing.

Legal foundation

The Law of the European Union applied.

Product support

In case of questions or difficulties concerning our instruments please contact your local distributor or G-FLEX directly during regular working hours.

Times available: Monday to Friday; 9am to 5pm (MET)

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G-Flex USA, Inc., 74 Southwood Pkwy Suite 600