**EN - ENDOSCOPE VALVES SET**

**FR - ENSEMBLE DE VALVES D'ENDOSCOPE**

**NL - ENDOSCOOPKLEPPEN INGESTELD**

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**Current Name:** IFU-Endoscope Valve Set


**Creation Date:** 19/05/2020

**Version** 1


*This document is only controlled in electronic format by the QM.*

EN - INSTRUCTION FOR USE .....	5
FR - MODE D'EMPLOI .....	7
NL - GEBRUIKSAANWIJZING .....	9
ES - INSTRUCCIONES DE USO .....	11
DE - GEBRAUCHSANWEISUNG.....	14
PT - INSTRUÇÕES DE USO.....	16
IT - ISTRUZIONI PER L'USO .....	18
EL - ΟΔΗΓΪΕΣ ΧΡΪΗΣΗΣ .....	21
AR العربية - تعليمات الاستخدام.....	23
TR - KULLANIM TALIMATLARI .....	25


**Fig. 1 - Olympus**

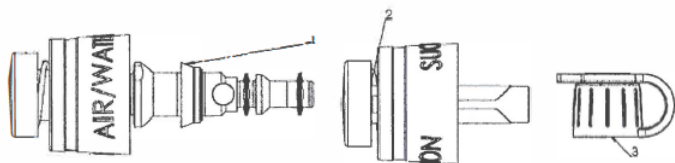
**Fig. 2 - Fujinon - 500 & 600 Series**

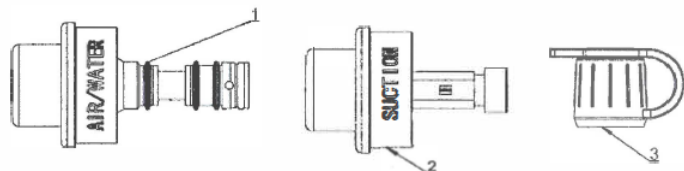
**Fig. 3 - Pentax**

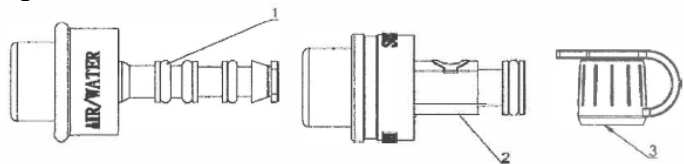**Fig. 4 - Fujinon - 700 Series**

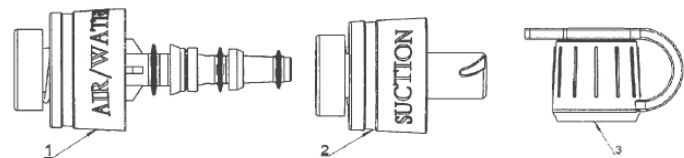**Fig. 5**
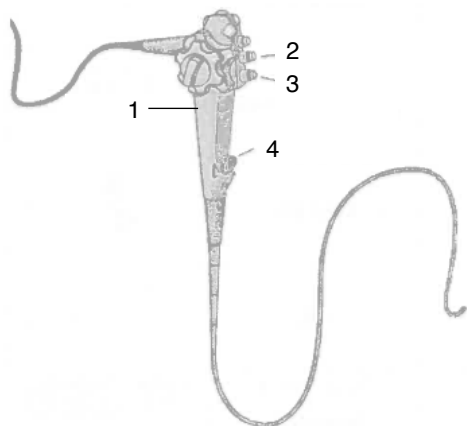
# EN - INSTRUCTION FOR USE

## Intended Use

The Endoscope Valves Set is used to fit into the air/water, suction and biopsy valves of an endoscope. The air/water valve is used to control the flow of gases and water. The suction valve is used to enable suction while preventing air inflow. The biopsy valve is intended to maintain the insufflation and minimize the leakage of material by sealing the biopsy channel.

## Product Description

The set is made of 3 valves as per the figures on page 3.

1. Air/water valve
2. Suction valve
3. Biopsy valve

Figure	Endoscope Brand	Endoscope Model
1	OLYMPUS	140/160/180/240/260/290 Series
2	FUJINON	500 and 600 series
3	PENTAX	i10 and 90 series
4	FUJINON	700 series

## Advices before first application

Please read carefully and follow all safety operating instructions and warnings before first application of the device.

A previous knowledge regarding handling and operation is required and essential. Unpack the instrument carefully and examine the devices for any possibility of damage. In case of any damage or missing items contact your distributor immediately.

## Handling and Operation

Ensure that the set of valves you have is compatible with the endoscope brand and model you are using.

## Procedure

- Assemble the valves into their respective port of the endoscope (see figure 5).
  1. Endoscope
  2. Suction valve
  3. Air/water valve
  4. Biopsy valve

## Storage

Do not put any objects on the instrument or its package!  
Do not store the instruments near aggressive chemical products!

Do not expose the instruments to direct or indirect sunlight or other ultra-violet rays!

Keep in dry area

Complaints will not be taken under consideration if the instruments have been stored improperly.

## Cleaning and Sterilization

This device is designed and warranted for single use!

## Risk in case of reuse

This device is for single use only. G-Flex denied all responsibility in case of reuse. The reuse of single use device might present risk to the safety of the user or patient, due to possible uncontrolled contamination and / or unreliability of the device performance.

## User

The users of G-FLEX instruments must be specialists in their fields. An appropriate and specific training for preparation, care and maintenance of the flexible instruments is required.

## Repairing and complaints

In health protection of our employees only disinfected or sterilized instruments will be accepted for analysis or repairing. In any case disinfection or sterilization dates and validity must be labelled outside the package. If this requirement has not been fulfilled the instruments will be returned to the sender without analysis or repairing.

## Product disposal

After use, these products may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

## Legal foundation

The Law of the European Union applied.

## Product support

In case of questions or difficulties concerning our instruments please contact your local distributor or G-FLEX directly during regular working hours.

**Times available:** Monday to Friday; 9am to 5pm (MET)

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