**INFLATION DEVICE  
DISPOSITIF DE GONFLAGE  
INFLATIE APPARAAT  
DISPOSITIVO DE INFLACIÓN  
INFLATION GERÄTE**

EN - INSTRUCTION FOR USE .....	2
FR - MODE D'EMPLOI .....	4
NL - GEBRUIKSAANWIJZING .....	6
ES - INSTRUCCIONES DE USO .....	8
DE - GEBRAUCHSANWEISUNG .....	11

**G-Flex Europe S.P.R.L.** - 20, Rue de l'industrie 1400  
Nivelles - Belgium | **Phone:** +32 67.88.36.65  
**Email:** [info@g-flex.com](mailto:info@g-flex.com) | **Website:** [www.g-flex.com](http://www.g-flex.com)  
**US Commercial Office Only:**  
G-Flex USA, Inc., 74 Southwood Pkwy Suite 600  
Atlanta, GA 30354 – USA

# EN - INSTRUCTION FOR USE

## Intended Use

Inflation devices are intended to inflate balloons for use in urological, gastrointestinal and biliary applications.

## Product Description

The device is made of an air or liquid syringe, with a lock lever mechanism allowing one-handed deflation, a pressure gauge for pressure measurement and a tubing with a luer-lock connector at the distal end to connect to the device to be inflated and a 3-way stopcock (if applicable).

## Advices before first application

Please read carefully and follow all safety operating instructions and warnings before first application of the device.

A previous knowledge regarding handling and operation is required and essential. Unpack the instrument carefully and examine the devices for any possibility of damage. In case of any damage or missing items contact your distributor immediately.

If the package present any damaged, the sterility of the device is not guarantee.

## Handling and Operation

Check to maximum volume of the inflator with the required volume of the device to be inflated. Verify the compatibility of the Luer-lock connection between the inflator and the device to be inflated prior to use.

Use the lock lever in the LEFT (unlocked) position for all aspiration and injection maneuvers.

Use the lock lever in the RIGHT (locked) position for pressure adjustment by rotating the palm grip on the piston.

## Procedure

- With the lock lever pushed LEFT aspirate enough liquid from a sterile bowl to fill approximately three fourths of the syringe capacity.
- With the Stopcock closed and the gauge inverted withdraw the plunger to evacuate any trapped air in the system.
- Open stopcock, hold device upright, tap barrel lightly in the hand and advance the plunger to release and purge any trapped air. Leave syringe approximately one-half full of contrast.
- Push the lock lever to the RIGHT into its upright position to engage the lock mechanism. Rotate the palm grip clockwise until the tubing is clear of air. Attach the tubing to the purged balloon catheter.
- To inflate, rotate the palm grip clockwise to the desired pressure.

- To gradually deflate, leave lock lever in upright engaged position and rotate palm grip counterclockwise until desired deflation pressure is reached. To rapidly deflate, push the lock lever LEFT and pull back plunger.

## Storage

This device is delivered sterilized and in order to maintain this state until its first use it must be kept in its original sterilized package.

Do not put any objects on the instrument or its package!  
Do not store the instruments near aggressive chemical products!

Do not expose the instruments to direct or indirect sunlight or other ultra-violet rays!

Keep in dry area

Complaints will not be taken under consideration if the instruments have been stored improperly.

## Cleaning and Sterilization

This device is designed and warranted for single use!

## Risk in case of reuse

This device is for single use only. G-Flex denied all responsibility in case of reuse. The reuse of single use device might present risk to the safety of the user or patient, due to possible uncontrolled contamination and / or unreliability of the device performance.

## User

The users of G-FLEX instruments must be specialists in their fields. An appropriate and specific training for preparation, care and maintenance of the flexible instruments is required.

## Repairing and complains

In health protection of our employees only disinfected or sterilized instruments will be accepted for analysis or repairing. In any case disinfection or sterilization dates and validity must be labelled outside the package. If this requirement has not been fulfilled the instruments will be returned to the sender without analysis or repairing.

## Legal foundation

The Law of the European Union applied.

## Product support

In case of questions or difficulties concerning our instruments please contact your local distributor or G-FLEX directly during regular working hours.

**Times available:** Monday to Friday; 9am to 5pm (MET)

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