EN - DISPOSABLE CLEANING BRUSH
FR - BROSSE DE NETTOYAGE JETABLE
NL - WEGWERP-REINIGINGSBORSTEL
ES - CEPILLO DE LIMPIEZA
DESECHABLE
DE - REINIGUNGSBÜRSTE FÜR DEN
EINMALGEBRAUCH
PT - ESCOVA DE LIMPEZA
DESCARTÁVEIS
IT - SPAZZOLA PER LA
PULIZIAMONOUSO
EL - ΒΟΥΡΤΣΑ ΚΑΘΑΡΙΣΜΟΥ Μ'ΙΑΣ
ΧΡ'ΗΣΗΣ
AR - فرشاة تنظيف يمكن التخلص منها
TR - TEK KULLANIMLIK TEMIZLIK
FIRÇASI

G-Flex Europe S.P.R.L. - 20, Rue de l'industrie 1400
Nivelles - Belgium | Phone: +32 67.88.36.65
Email: info@g-flex.com | Website: www.g-flex.com

EN - INSTRUCTION FOR USE	3
FR - MODE D'EMPLOI	4
NL - GEBRUIKSAANWIJZING	7
ES - INSTRUCCIONES DE USO	9
DE - GEBRAUCHSANWEISUNG	11
PT - INSTRUÇÕES DE USO	13
IT - ISTRUZIONI PER L'USO	15
EL - ΟΔΗΓΊΕΣ ΧΡΉΣΗΣ	17
AR - العربية - تعليمات الاستخدام	19
TR - KULLANIM TALIMATLARI	21

EN - INSTRUCTION FOR USE

Intended Use

The Cleaning Brush is meant to clean the biopsy channel of endoscopes or the endoscope valves. The Sponge is intended to clean the outer part of the endoscope or protect the endoscope during transportation.

Product Description

The device is made of a long catheter (or spiral) with a brush on each side.

Advices before first application

Please read carefully and follow all safety operating instructions and warnings before first application of the device.

A previous knowledge regarding handling and operation is required and essential.

Unpack the instrument carefully and examine the devices for any possibility of damage. In case of any damage or missing items contact your distributor immediately.

Handling and Operation

The Cleaning Brush has to be passed through the full length of the endoscope biopsy channel several times with the solution described in your endoscope manufacturer notice.

Procedure

1. Introduce the brush through one side of the biopsy channel and pull it out from the other side of the same.
2. Repeat this process until the working channel is free from dirt. (Recommended a minimum of 3 times)
3. Ensure to clean the brush before introducing it in the channel each time.

Please note that some models have the distal and proximal ends different from each other. In such models, make sure to introduce the brush with the thinnest diameter tip into the channel.

Storage

This device is delivered sterilized and in order to maintain this state until its first use it must be kept in its original sterilized package.

Do not put any objects on the instrument or its package!
Do not store the instruments near aggressive chemical products!

Do not expose the instruments to direct or indirect sunlight or other ultra-violet rays!

Keep in dry area

Complaints will not be taken under consideration if the instruments have been stored improperly.

Cleaning and Sterilization

The disposable version of this device is designed and warranted for single use!

Risk in case of reuse

The Cleaning Brush is for single use only. G-Flex denied all responsibility in case of reuse. The reuse of single use device might present risk to the safety of the user or patient, due to possible uncontrolled contamination and / or unreliability of the device performance.

User

The users of G-FLEX instruments must be specialists in their fields. An appropriate and specific training for preparation, care and maintenance of the flexible instruments is required.

Repairing and complains

In health protection of our employees only disinfected or sterilized instruments will be accepted for analysis or repairing. In any case disinfection or sterilization dates and validity must be labelled outside the package. If this requirement has not been fulfilled the instruments will be returned to the sender without analysis or repairing.

Product disposal

After use, these products may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

Legal foundation

The Law of the European Union applied.

Product support

In case of questions or difficulties concerning our instruments please contact your local distributor or G-FLEX directly during regular working hours.

Times available: Monday to Friday; 9am to 5pm (MET)

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