EN - FOREIGN BODY RETRIEVER

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ÉTRANGERS

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EN - INSTRUCTION FOR USE

Intended Use

Foreign Body Retrievers are intended for taking out foreign bodies from the human body through endoscopes.

Product Description

The Foreign Body Retrievers are made of a flexible tube with a grasping distal end and with or without a proximal handle.

Advices before first application

Please read carefully and follow all safety operating instructions and warnings before first application of the device.

A previous knowledge regarding handling and operation is required and essential. Unpack the instrument carefully and examine the devices for any possibility of damage. In case of any damage or missing items contact your distributor immediately.

Unwind the Foreign Body Retrievers, manipulate backward - forward the plastic spool of the handle to verify the opening / closing of the cups assembling

Considering that device are not delivered sterilized the user must do it before the first application (See Cleaning and Sterilization)

If the package present any damaged, the sterility of the device is not guarantee.

Handling and Operation

The device must be inserted through the endoscope operating channel with its graspers withdrawn into the tube. Please observe compatibility between Foreign Body Retrievers and the endoscope operating channel diameter.

As some devices are delivered without handle, before inserting the device into the operating channel, assemble it to the universal handle. During the handle assembling the device cups must be closed.

Foreign Body Retrievers are composed by a spiral and should never be manipulated with its spiral winded as this may damage the device and make its usage impossible!

Procedure

Once the foreign body is located in the patient's body the following procedure should be done:

- Insert the device into the endoscope.
- Place the distal end of tube in front of the object to be removed.
- Push the handle spool in order to take the grasping end out of the tube and then bring the object to be captured in the center of the graspers.

- Close the prongs carefully as much as the object allows by pulling back the handle spool.
- Remove the object and the grasper gently withdrawing the endoscope. Always keep the grasper prongs tightly closed.

Contraindication

- Bleeding disorders
- Tissue samples from stumps in vascular ulcers
- Inadequate preparation of the patient (intestinal)
- General. OGD, colonoscopy, proctoscopy, sigmoidoscopy

Possible complication

- Perforation, bleeding, difficulties to remove/insert, hemorrhage, infection, sepsis, allergic reaction to medication, hypotension, respiratory depression or arrest, cardiac arrhythmia or arrest.

Storage

Do not put any objects on the instrument or its package!
Do not store the instruments near aggressive chemical products!

Do not expose the instruments to direct or indirect sunlight or other ultra-violet rays!

Keep in dry area

Complaints will not be taken under consideration if the instruments have been stored improperly.

Cleaning and Sterilization

G-Flex recommend to follow one of the following sequences when sterilizing the device:

- Manual Cleaning, Ultrasonic Cleaning, Automatic/Thermal Cleaning and Autoclave; or
- Manual Cleaning, Ultrasonic Cleaning, Automatic/Thermal Cleaning and Ethylene Oxide Gas Sterilization

Either sterilize by Autoclave or by ETO. There is no need to do both.

Manual Cleaning

The cleaning of the device should be done immediately after each use:

- Immerse the device in a suitable liquid detergent or disinfectant (Always observe the minimum diameter of 20 cm when winding the instrument and applicability of the cleaning product!).
- Wipe the instrument with a soft towel.

Ultrasonic Cleaning

The ultrasonic cleaning facilitates the dislodging of residual material and must be done before sterilization. The ultrasonic cleaning must last for at least 30 minutes.

- Clean the device right after its use.
- Put the instrument in an ultrasonic cleaning tank always observing the winding minimum diameter 20 cm (Please observe the ultrasonic equipment manufacturer's instructions!).
- Use only tap water and the detergents which dissolve albumen, always according to ultrasonic equipment manufacturer's instructions.
- Rinse the instrument with tap water and wipe dry by using gauze pads.

Automatic / Thermal Cleaning

Alternatively or additionally to the manual cleaning, this instrument may also be cleaned in automatic / thermal cleaning machines by using approved alkaline detergents. Please refer to relevant instructions of the cleaning machine supplier.

Sterilization - Autoclave

Before autoclaving sterilization the instrument should be cleaned carefully as described in Manual Cleaning, Ultrasonic Cleaning and Automatic/Thermal Cleaning.

This symbol means that the product can be sterilized by autoclave according to the below parameters

Parameters:

Temperature:	134 °C / 273 °F
Pressure:	3 bar
Exposure:	≥ 18 min

Also in here the minimum 20 cm diameter for winding the instrument must be noted.

The original packages of G-FLEX are not autoclavable!

Sterilization - Ethylene Oxide Gas Sterilization

Please follow manufacturer's warnings regarding to the biological indicators

Parameters:

Temperature:	57 °C
Pressure:	1,7 bar
Exposure:	max. 4 h
Relative Humidity:	50 %
Gas Concentration:	12 %
Aeration Time:	7 days in ambient temperature or 12 hours by 50-57 °C

User

The users of G-FLEX instruments must be specialists in their fields. An appropriate and specific training for preparation, care and maintenance of the flexible instruments is required.

Repairing and complaints

In health protection of our employees only disinfected or sterilized instruments will be accepted for analysis or repairing. In any case disinfection or sterilization dates and validity must be labelled outside the package. If this requirement has not been fulfilled the instruments will be returned to the sender without analysis or repairing.

Product disposal

After use, these products may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

Legal foundation

The Law of the European Union applied.

Product support

In case of questions or difficulties concerning our instruments please contact your local distributor or G-FLEX directly during regular working hours.

Times available: Monday to Friday; 9am to 5pm (MET)

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